

How to replace MethaMeasure tubing.

Before starting to change the tube please prime any liquid out of the pump. Make sure you only use tube supplied by MethaMeasure Ltd.

1 - Unlock the cabinet door using the key.



2 - Remove the tubing from the Methadone bottle.



3 – Then flip the lever on the motor head from the right to the left. This will release the tubing and any liquid that is possibly still stuck in the tubing.



4 – Once the motor head is fully open, remove the tubing from the white connector and you will now be able to slide the tubing out the motor head





5 - Remove the tube from each clip on both sides of the motor and pull through the hole.




6 - Using the new connector received push it through the left hole until it is secure in the hole. The connector should now be showing at both sides. Attach the shorter stiffer PVC tube to the bottom half of the connector.



7 - Place the new silicone tube back into the left and right clips of the motor head. Adjust the tube by pulling either side gently. The tube should be sat on the chrome rollers.




8 - Attach the connector and weight back onto the tube ends.

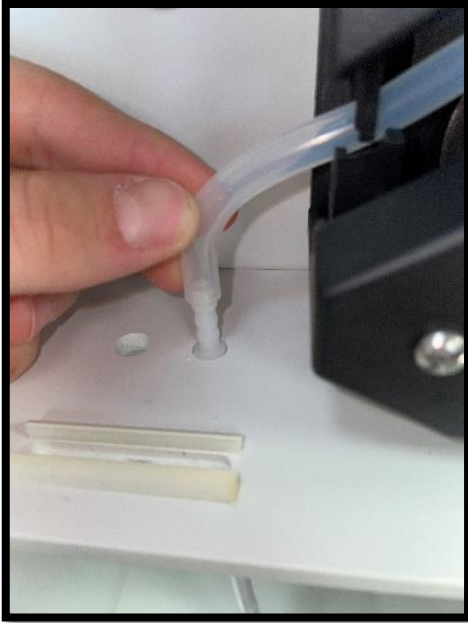

9 - Return tube back into the methadone bottle and lock the cabinet. The pump can now be reprimed and must be recalibrated before any dispensing takes place. Please call 0800 027 0671 Option 1 to confirm the tube change has taken place.



DOCUMENT DETAILS

 METHA MEASURE LIMITED	Tube Change Guide		
	No: MAN002	Rev: 0	Page: 10/10

REVISION HISTORY

Rev	CR #	Approval / Effective Date	Description of Change	Originator
0	CR-2023-07	3/10/2023	Release documentation	C Cave